Quickstart **GUIDE**

www.elcie-healthy.com



1 CHARGE YOUR CONNECTED GLASSES

Charge your connected glasses until the green LED is blinking (full charge).


The charger of your connected glasses is composed of:

- **A USB power cable**
- **A charging magnetic connector**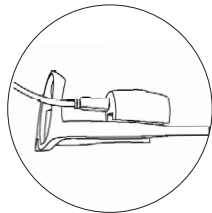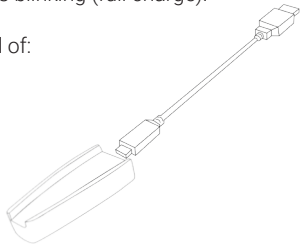

2 ADAPT THE GLASSES' ARMS.

The end of branches are adjustable. Bend them gently in order to fit your ears.

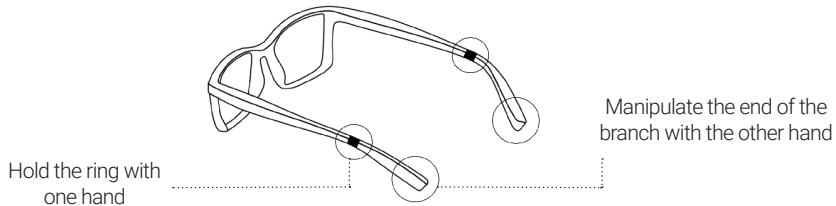3 INSTALL THE ELLCIE HEALTHY APP

Download the Elcie Healthy application on the Store, by typing « Elcie Healthy » in the search bar.

Run the application and accept the permissions required by the application.

Create your account as a new user and register by agreeing the terms of service.

4 CONNECT YOUR GLASSES WITH YOUR SMARTPHONE

After completing your registration and confirming the creation of your account, the application takes you to the «START PAIRING».

Fold the branches of your glasses and take them in one hand. **Shake the glasses** vigorously for 5 to 10 seconds until you hear the **pairing melody** and see the LEDs light up in blue. On your App, wait during the search and **click on the line** corresponding to your glasses.

5 USE CASE

Drowsiness detection

- During the trip, the application informs you in real-time of your **level of drowsiness**. If there is any sign of fatigue, the glasses assess the risk and **send you light and sound signals**.
- In the event of a level 5 alert, a **teleoperator*** will offer you personalized support to ensure your safety and you will benefit from the following Tele assistance services.
- In case of drowsiness detected, your loved ones are **alerted** in real-time by SMS.


Fall detection

- When worn and connected via Bluetooth with your smartphone, the connected glasses automatically trigger an SOS alert.
- If a fall is detected, an alert will be sent to your family or a professional helpline to provide the user with appropriate assistance as soon as possible.
- Manual triggering of an alert from the Serenity by Ellcie Healthy mobile application allows you to send a message to the designated person.

6 HOW DOES IT WORK?

Fifteen embedded sensors (accelerometer, gyroscope, temperature sensor, atmospheric pressure ...), collect physical, physiological and environmental data.

Connected Eyewear features the latest in state-of-the-art technology. Unique Artificial Intelligence analyzes the collected data. A specialized learning system refines this information and provides accurate, real-time results. to reality as possible.

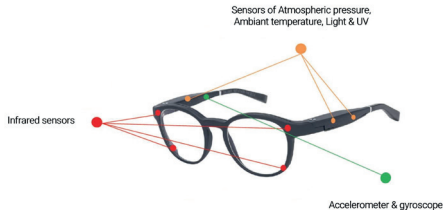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Download the complete user guide

An abnormal use of your Ellcie Healthy connected glasses (non-compliance of the recommended uses as presented below) may void the Ellcie Healthy warranty. Without being exhaustive, do not heat the glasses or put them near a heat source, do not immerse the glasses under water, do not step on the glasses, do not have children (or animals) wear the glasses, do not put the glasses in a microwave. For an optimal performance, make sure to have a good centering of the glasses on your face. Some make ups could also interfere with the sensor's behavior.

In order to ensure the optimal use of your connected glasses, personal data, including physiological data that may constitute personal health data, may be collected by the Ellcie Healthy app. Using Ellcie Healthy application implies your consent to this collection. For more information about collected data and your rights in relation to the processing of this data, you can visit our website at the following address:
www.ellcie-healthy.com/legal-notice/