# Quickstart **GUIDE**

[www.elcie-healthy.com](http://www.elcie-healthy.com)

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## Read and keep the safety instructions & the user manual

### 1 Charge your connected glasses

Charge your connected glasses until the green LED is blinking (full charge). The charger of your connected glasses is composed of a USB power cable and a charging magnetic connector.

### 2 Adapt the glasses' arms

The sleeves at the end of branches are adjustable. Bend them gently in order to fit your ears. Hold the ring with one hand and manipulate the end of the branch with the other hand.

### 3 Install the Ellicie Healthy App

Download the Ellicie Healthy application on the store by typing «Ellicie Healthy» in the search bar. Run the application and accept the permissions required by the application. Create your account as a new user and register by agreeing the terms of service.

### 4 Connect your glasses with your smartphone

After completing your registration and confirming the creation of your account, the application takes you to the «start pairing». Fold the branches of your glasses and take them in one hand. Shake the glasses vigorously for 5 to 10 seconds until you hear the pairing melody and see the LEDs light up in blue. On your App, wait during the search and click on the line corresponding to your glasses.

### 5 Use cases

After completing your registration and confirming the creation of your account, the application takes you to the «start pairing». Fold the branches of your glasses and take them in one hand. Shake the glasses vigorously for 5 to 10 seconds until you hear the pairing melody and see the LEDs light up in blue. On your App, wait during the search and click on the line corresponding to your glasses.

### 6 How does it work ?

When you wear the glasses, the sensors embedded in the frame collect physical, physiological and environmental data. Falling asleep as well as the fall are accompanied by physiological signs that are captured by the glasses. Thanks to Artificial Intelligence and Machine learning (self-learning system), the efficiency of the glasses is increased as they are used. In short, the more you wear the glasses, the better they will protect you.

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